

Quick Start - Link Drench Gun to iScale

1.

Set up the iScale

- » On the Te Pari weigher push the **Setup/ Weigh** button.
- » On the first page (SETUP PAGE 1 OF 1)
 - set Current Favorite to **SHOWDRE**
- » The settings should be as shown...

SETUP PAGE 1 OF 3

CURRENT FAVORITE:	SHOWDRE
SAVE/NAME FAVORITE:	SHOWDRE
BACKLIGHT MODE:	AUTO
SET DRAFT TYPE:	WEIGHT
DRAFTING SETUP:	
HEALTH MODE:	HEALTH 1 ON
SET HEALTH 1 TYPE:	IVOMEK
CALCULATED DOSE:	10ML / 0010KG
SET HEALTH 2 TYPE:	DRENCH
CALCULATED DOSE:	10ML / 110KG
WEIGHT RECORD:	AUTO
AUTO ZERO:	ON
WEIGHT GAIN:	GAIN/DAY
TARE WEIGHT:	000.0 KG

SETUP PAGE 2 OF 3

WIFI/COM 1 MODE:	EID READ / PC CONNECT
WIFI/C1 BAUDRATE:	9600
COM 1 POWER:	OFF
COM PORT 2 MODE:	SIMPLE WEIGHT
COM 2 BAUDRATE:	9600
WIFI/C2 POWER:	ON
BLUETOOTH MODE:	EID READ / PC CONNECT
BLUETOOTH POWER:	ON
SYSTEM SETUP:	
SYSTEM INFORMATION:	

SETUP PAGE 3 OF 3

ID MODE:	NO ID QUICK WEIGH
ENABLE AUTO VID:	OFF
AUTO VID START:	VID001
ENABLE PID:	NO
PID DEFAULT:	
ENABLE CODE 1:	NO
CODE 1 TITLE:	CODE 1
CODE 1 DEFAULT:	
ENABLE CODE 2:	NO
CODE 2 TITLE:	CODE 2
CODE 2 DEFAULT:	
ENABLE CODE 3:	NO
CODE 3 TITLE:	CODE 3
CODE 3 DEFAULT:	
LOAD CODES WITH PREV WEIGHT:	NO

2.

Set up the drench gun

The drench gun has two modes - AP mode (Access Point for WIFI) for when it is linked with the Stock Recorder, and DH mode for when it is just linked to a weigh head.

Change the drench gun to DH mode:

- » Turn the drench gun on by pressing and releasing the power button.
- » Press the menu button and step through to the WiFi menu.
- » Ensure WiFi is on.
- » While in this screen press the **Stats** and **Prime** button at the same time
- » The screen will display either "ap" or "dh". You need it to be "dh".
- » If it is "ap" change it to "dh". Press the up arrow to change the menu to "dh"
- » Press the menu button to confirm the change. Wait for the progress bar to finish. Turn off the drench gun by pressing and holding the power button: then turn the drench gun back on again.

3.

Link up

- » On the Te Pari weigher push the **Setup/ Weigh** button to return to the weight screen.
- » The drench gun and iScale should link. The wifi symbol on the gun will appear.
- » Step on the scale to get a weight. The dose should appear on the drench gun screen.
- » If the wifi link keeps breaking then there is too much wifi in the area and you are screwed.